

Quick Start Guide



Trigno™ Avanti Mobile

NEED ASSISTANCE?

North America

SUPPORT

(508) 545-8200
support@delsys.com

TRAINING

(508) 545-8200
training@delsys.com

Europe

+44 161 743 3540
support@delsyseurope.com


+44 161 743 3540
training@delsyseurope.com




1 Experiment Set-up

Follow steps **A** to **D** to achieve high quality surface EMG signals.

- A** Remove sensors from charging cradle and touch 

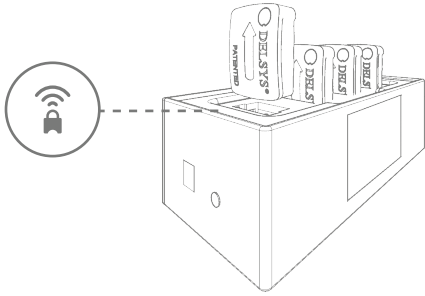

- B** Open the desired application from the Mobile EMG Suite in your Android tablet, with sensors nearby for auto-recognition.

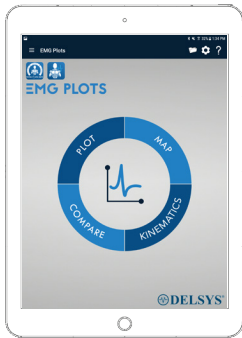


EMG Plots




EMG Reports

- C** Abrade and sanitize a shaved skin site.

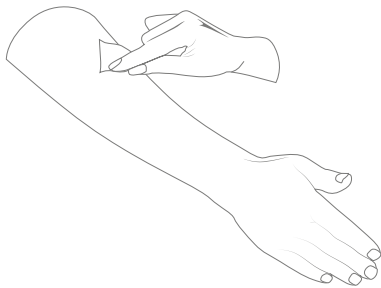


- D** Place Trigno™ sensors over muscles using double-sided adhesives.

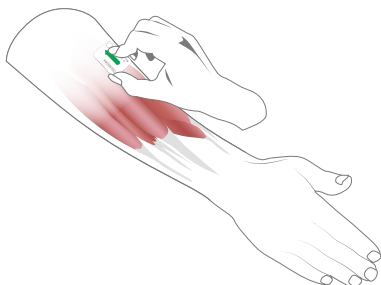




2 Record Data with Mobile EMG Suite

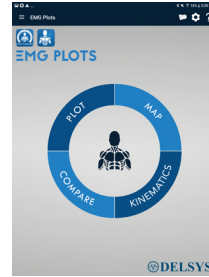- A** Select your preferred acquisition module.

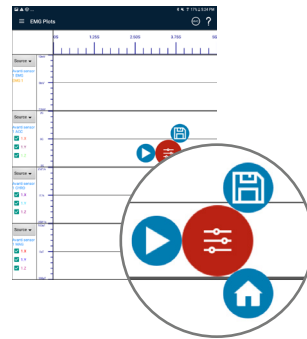

- B** Click  to show Start and Record buttons.




- C** Click  to start data stream. Click  to record data.



- D** Click **Save** to stop recording and save data. Click **Stop** to end data stream.



- E** Click the  icon in the top right, find your file and click to Export.

